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Practical Use of PPE in Pandemic COVID-19

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ANSI/AAMI PB70:2003, AAMI PB70:2012 , FDA 2020

Level	Risk	Fluid barrier	Test	Activity	Example
1	Minimal	Slight	Water impaction	Basic care, Standard hospital medical unit	Isolation Gown
2	Low	Splatter Some soaking	Add pressurizing the material 180-360 mmH ₂ O	Blood draw from a vein, Suturing, Intensive care unit, Pathology lab	Coverall-1
3	Moderate	> Level 2	As level 2, 360-720 mmH ₂ O	Arterial blood draw, Inserting an IV, Emergency Room, Trauma	
4	High	Prevents all fluid > 1 h and may for virus	As level 3, > 720 mmH ₂ O Blood + virus splash	Pathogen resistance, Infectious diseases (non-airborne), Large amounts of fluid exposure over long period	Coverall-2

Isolation Gown

Table 9. Recommended surgical gown fabrics for different AAMI protection levels.

AAMI protection	Surgical gown fabric
Level 1	35 g/m ² spunlace fabric without any fluorochemical finish
Level 2 Coverall-1	35 g/m ² spunbond fabric with 4% fluorochemical
Level 3	35 g/m ² SMS fabric with 4% fluorochemical
Level 4 Coverall-2	35 g/m ² SMS fabric with 4% fluorochemical and 1.5% antibacterial finish

AAMI: Association for the Advancement of Medical Instrumentation; SMS: spunbond/meltblown/spunbond.

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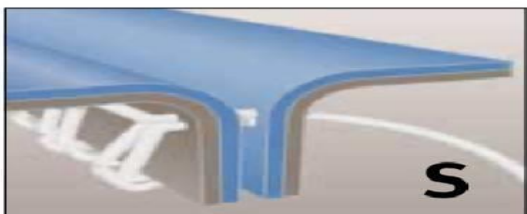
Seam construction

Seams are a critical component of the overall barrier protection provided by a chemical protective garment. It is vital to select the appropriate seam configuration for your application needs and to know that the garment will be constructed with strong, tight seams. One loose thread or gap and the barrier between you and your environment unravels—leaving you vulnerable.

Serged or Sewn*

A seam produced when multiple threads are interlocked around the raw edges of two pieces of material for a strong, stress-resistant seam.

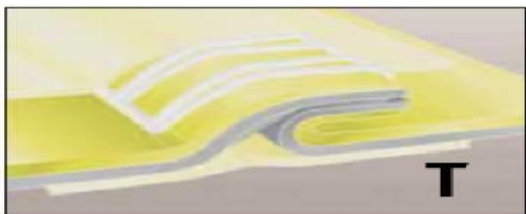
Bound*

Tightly sewn with a reinforced outer binding to increase seam strength and barrier. For potential misting exposure to non-hazardous liquids or particle penetration through the seam.

Taped

Both sewn and taped to provide strong chemical resistance against heavy liquid splashes and tough seam stress. A sewn seam is covered with a strip of compatible material by heat-sealing.

Double Taped

Sewn, then taped on the inside and the outside of the seam for a very strong chemical- and stress-resistant seam.

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Reusable Coverall

- **Made of polyester, safely laundered, do not touch outer surfaces during care**
- **Frequent inspect for a small hole, thin, or ripped**
- **Can be reused at least > 5 times or until present of defects**
- **Sterilization for killing of contaminated virus or bacteria before use**
 - **Autoclave at 121-134 °C for 20-40 min**
 - **Hot air oven 160-180 °C for 2 h**
 - **Wash in hot water 60-70 °C for 40 min**
 - **UVC or ozone**

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Surgical N95 Respirator (healthcare model)

([www.cdc.gov › topics › respirators › disp_part › n95list1sect2-m](http://www.cdc.gov/topics/respirators/disp_part/n95list1sect2-m))

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N95 Respirator (healthcare vs occupational model)

Division	Healthcare/ Medical	OH&ES	Healthcare/ Medical	OH&ES
Model/Catalog Number	1860/1860S	8210/8110S	1870	9210
				
NIOSH Approved	Yes	Yes	Yes	Yes
N95 Filtration	Yes	Yes	Yes	Yes
FDA Cleared	Yes	No	Yes	No
Fluid Resistant Claim	Yes	No	Yes	No
Can be used in surgery	Yes	No	Yes	No
Fit testing required when switching to or from the Healthcare or Occupational models	Yes	Yes 	No, when switching from 9210 based on identical construction*	No, when switching from 1870 based on identical construction*

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